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SPEIKO® 10% Citric Acid Solution

Composition:

Low-viscosity 10% solution of citric acid in purified water.

Indications:

Chemical root canal dilatation. Location of canal entrances. Citric acid is a mild, slightly antibacterial, biocompatible chelating agent that forms a relatively stable chelate complex with the calcium ions in the dentine (action similar to EDTA preparations). This demineralizes and softens hard dental substance and removes the smear layer on the canal wall, thus opening the dentine tubuli and even widening them. Residual citric acid must be thoroughly flushed out to prevent continuing demineralization along the length of the tubuli. Opening up the dentine tubuli facilitates adaptation of the root canal sealer, generally improving the sealing of the root canal filling. This effect is further enhanced by activating the solution with ultrasound. Calcium hydroxide residues can also be reliably removed (for instance from a medical inlay).

Counterindications:

Allergic reactions to citric acid.
Wide-open apical foramen.

Adverse reactions:

No adverse reactions are known following product use as intended. Irritation of periapical tissues can be expected if the material exits through the apex.

Interactions with other agents:

None known.

Dosage, mode and duration of application:

SPEIKO 10% Citric Acid Solution must be dosed under appropriately hygienic conditions.

In cases of root canal entrances that are hard to locate, SPEIKO 10% Citric Acid Solution can be dripped into the pulp cavum. The root canal entrances become visible after a few minutes.

The solution is injected into the root canal for preparation and dilatation using a pipette or suitable rinsing cannula. This should be repeated throughout the entire root canal preparation procedure. Alternate rinsing with sodium hypochlorite (Hypochlorit-SPEIKO®) to remove necrotic tissue, dentine residues and the smear layer, rinsing with water between applications. Following root canal preparation, remove SPEIKO 10% Citric Acid Solution completely by rinsing with sodium hypochlorite or a chlorohexidine solution (Speikosept CHX 2%). This is followed by drying of the root canal with sterile paper tips.

Notices:

This medicinal product must not be used after the expiry date.
Store out of reach of children.

Commercial product form: Liquid Contents: 30 ml (Art. No. 1465)
Liquid Contents: 100 ml (Art. No. 1466)

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SPEIKO®- Dr. Speier GmbH P.O. Box 7204 D-48039 Münster