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Please read carefully

## CALASEPT - SPEIKO® including X-ray contrast

### Composition:

Contains calcium hydroxide in an aqueous suspension, barium sulphate, calcium chloride, potassium chloride, sodium chloride, sodium hydrogen carbonate, purified water.

### Applications:

Treatment of non-cariou dentine adjacent to pulp, deep carious lesions or indirect pulp capping, direct pulp capping, also following pulpotomy and as a temporary root canal inlay.

### Contraindications:

Exposed pulp in carious dentine. Irreversibly diseased pulp. Allergy to the ingredients.

### Side-effects:

During application to exposed pulp, the strongly alkaline properties may cause pain reactions.

### Interactions with other agents:

In case of direct pulp capping, calcium hydroxide neutralises the effect of acid-containing liners.

### Dosage instruction, form and duration of application:

#### Pulp capping (direct and indirect):

Using the (curved) Calasept needle supplied, small quantities of paste are placed on a mixing block and from there applied selectively to the exposed pulp or dentine adjacent to the pulp using an instrument. After application, the paste should be tamped using water-absorbing materials (pledgets of cotton or paper points). It is then sealed with a liner.

#### Temporary root filling:

As a temporary filling material for root canals, Calasept can be applied direct to the prepared canal using, for example, a lentula. The paste can also be applied to the root canal using the disposable needles supplied. If necessary, the application can be repeated. The calcium hydroxide paste should be removed before final root filling.

#### Treatment of deep carious lesions:

Before applying a thin coating of Calasept, clean and drain cavity. If a thicker coating is desired, apply in layers. Briefly dry each layer in a gentle stream of air. Then insert liner and filling.

### Use of correct needle:

The (curved) Calasept needle has a particularly sharp tip on the side that perforates the rubber membrane of the Calasept ampoule, in order to keep the hole in the membrane small. **Please note: After using the Calasept needle as well as the disposable needles, they have to put in the added rubber stopper strictly. This prevents hardening of Calasept due to air contact.**

### Use of correct carpule syringe:

We recommend the use of a 'carpule' syringe (Heraeus Kulzer). The syringe used for Calasept-SPEIKO® must have a well-centred plunger that cannot move sideways. Syringes with aspirator hooks or V-shaped plungers must not be used.

### Special instructions:

Calcium hydroxide causes local irritation and corrosion to the skin, eyes, and mucous membranes. Particles of calcium hydroxide in contact with the skin or eyes must be mechanically removed by appropriate means, then rinsed immediately and thoroughly with water. Do not use this medical product after the expiry date. Keep this medical product out of reach of children.

### Available in the following packs:

2 barrel ampoules containing 2.5 g paste and accessories  
25 disposable needles

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