SPEIKO® EDTA - Lösung

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Fields of application:

Chemical expansion of the root canal. Identification of canal openings. Sodium edetate forms a relatively stable chelate complex with the calcium ions of dentin. The tooth's hard substance is demineralised and softened in the process which also removes the smear layer adherent to the wall of the canal. The dentin tubuli are opened and, if possible, extended. In order to prevent progressive demineralisation alongside the tubuli, residual EDTA must be rinsed out thoroughly. The opening of the dentin tubuli improves the adaptation of the root canal sealant which induces an improvement of the root canal's sealing altogether.

Contraindications:

Allergic reactions to edetic acid/ sodium edetate.
Wide open foramen apicale.

Side-effects:

No side-effects are known if properly applied. Irritation of periapical tissue is likely to occur if the material reaches over the apex.

Interactions with other substances:

Unknown.

Dosage instructions, form and duration of application:

Comply with hygienic conditions whilst removal of EDTA - solution.

SPEIKO® EDTA-Solution can be dripped into the pulp cavity to locate hard-to-find root canal openings. The root canal openings appear after several minutes.

In order to prepare and extend root canals, the solution is applied into the root canal by means of pipette or a cannula suitable for rinsing. The procedure should be repeated throughout the whole process of root canal preparation. Rinse alternately with sodium hypochlorite (Sodium Hypochlorite SPEIKO®) in order to remove pulp remnants, necrotic tissue, dentin residues, and the smear layer. After preparation of the root canal has been completed, the EDTA -Solution should be completely removed from the root canal by rinsing with sodium hypochlorite solution or a solution of chlorohexidine (Speikosept CHX 2%). Drying of the root canal should then proceed by applying sterile paper tips.

Special instructions:

This medical product must not be used after its shelf life has expired.

This medical product is only suitable for dental use; there is no experience available in other areas of application.

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SPEIKO®- Dr. Speier GmbH P.O.Box 7204 D-48039 Münster