



## **SPEIKOSEPT CHX 2 %-Gel**

**Zusammensetzung:** Gel enthält 2 % Chlorhexidindigluconat

**Anwendungsgebiete:**

Zur Reinigung kontaminierter Wurzelkanalinstrumente während der Wurzelkanalbehandlung.

**Dosieranleitung, Art und Dauer der Anwendung:**

Eine ausreichende Menge des Gels in ein geeignetes Gefäß (Arbeitsstände für Endofeilen) geben. Die gebrauchten Instrumente werden nach der Verwendung in das Gel eingebracht. Auf Grund der bakteriziden Wirkung des Chlorhexidindigluconats ist von einer raschen Keimzahlverminderung auszugehen. Der Tensidzusatz des Produkts gewährleistet eine rasche und vollständige Ablösung behandlungsbedingter Rückstände vom Instrument. Ein Abspülen der Instrumente ist nicht erforderlich. Vor der Sterilisation empfiehlt sich eine weitere Aufbereitung der Instrumente.

**Hinweise:**

Dieses Medizinprodukt darf nach Ablauf des Verfallsdatums nicht mehr angewendet werden. Nach Gebrauch Spritze wieder dicht verschließen und bei Raumtemperatur lagern. Unzugänglich für Kinder aufbewahren.

**Handelsform:**

Speikosept CHX 2 %-Gel      Inhalt: 3,0 g Art.-Nr. 1071

Stand der Information: Dezember 2010

---

**SPEIKO® - Dr. Speier GmbH Postfach 7204 D-48039 Münster**  
Telefon 0251/789040 Telefax 0251/7890422 E-Mail: [info@speiko.de](mailto:info@speiko.de) [www.speiko.de](http://www.speiko.de)

Printed in Germany Imprimè en Allemagne



## **SPEIKOSEPT CHX 2 %-Gel**

**Composition:** Gel contains 2 % chlorohexidin-digluconate

**Applications:**

Cleansing of contaminated root canal instruments during treatment of root canals

**Dosage instructions, form and duration of application:**

Pour a sufficient amount of gel into an appropriate vessel (endo tray boxes). Immerse used instruments in the gel after use. A reduction of the number of bacteria can be anticipated on account of the bactericidal mode of action of chlorohexidin-digluconate. The addition of surfactants to the product ascertains that remnants adhering from treatment are rapidly and completely detached from the instruments. Subsequent rinsing of the instruments is not necessary. Additional preparation of the instruments before sterilising is recommended.

**Special instructions:**

This medical product must not be used after its shelf-life has expired.  
Close syringe tightly after use and store at room-temperature.  
Keep this medical product out of the reach of children.

**Available in the following packs:**

Speikosept CHX 2%-Gel          contents: 3 g    Article-No. 1071

Status: december 2010

