EN ISO 1567, Typ 1, Gruppe 1

KALLOCRYL®

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hot cure types

Composition:

Powder contains polymethylmethacrylate, benzoyl peroxide, artificial colour (cadmium-free)
Liquid contains methylmethacrylate, aromatic substances

Applications:

manufacturing of full and partial prosthetics for condensing and pressure-moulding, in particular, prosthetics for which high pressures are assumed to persist on account of unfavourable circumstances..
Attention: this synthetic material **cannot** be cast!

Contraindications:

Direct denture relining. Contact of unpolymerised material with the oral mucosa.
Hypersensitivity or allergic reactions to ingredients.

Side effects / interactions: Unknown.

Dosage instructions, form and duration of application:

Mixing two mass proportions of powder with one liquid proportion is recommended. The composition of the mixture may be varied to a minor extent. The liquid is first given into the blending vessel made of inert material, then rapidly add the powder and blend thoroughly using a spatula. Avoid producing air bubbles. The covered-up cake is left to stand until it can be drawn from the wall of the vessel without filaments. 4-6 minutes will remain for further processing. The cuvettes are compressed in a pressure chamber. A pressure of approximately 4 bar is recommended. heating up to 100 °C in 50 minutes is the optimum for controlling the polymerisation reaction. Keep the temperature constant for 50 minutes and then cool down to ambient temperature. Alternative processing at 80 °C: Here, heat up rapidly to 70-75°C. For this purpose the cuvette may be placed in a pre-heated water-bath kept at this temperature. Hold the temperature for 30 minutes, then increase to 80-85°C. This temperature should be held constant for 150 minutes, then allow the cuvette to gradually cool down to ambient temperature.

Storage:

Reseal powder tin-box properly after use. Avoid exposure to moisture. Reseal the liquid bottle immediately after use. Do not store above 25°C, store cool and in the dark. Never expose to direct sunlight, as this may induce spontaneous hardening. Do not use after shelf-life expires.

Waste disposal:

Empty bottles can be recycled. Take up liquid residues using adsorbent agents and then dispose as regular waste. Tins containing powder rests can be disposed as regular waste.

Available in the following packs:

Kallocryl® B	light pink	tin: 500 g	article no.: 1648
Kallocryl® B II	pink	tin: 500 g	article no.: 1654
Kallocryl® B	liquid	bottle: 250 ml	article no.: 1664

Notice: The colour of Kallocryl® B II is compatible with the cold polymerisate A II.

Special warnings:

R 11:	Liquid is highly inflammable (classification A I acc. to Inflammable Liquid Ordinance).
R 36/37/38	Irritates eyes, airways and skin.
R 43	Potential sensibilization due to skin contact.
S 2	Must be kept out of the reach of children.
S 9	Keep container at a well ventilated place.
S 16	Keep away from light sources – do not smoke.
S 24	Avoid skin contact.

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