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Stand der Information: November 2005

SPEIKOCAL with radiopaque medium

Composition: Contains calcium hydroxide and barium sulphate in an aqueous suspension.

Indications: Treatment of non-carious dentine adjacent to pulp, deep caries lesions or indirect pulp capping, direct pulp capping following pulpotomy and as a temporary root canal inlay.

Contraindications: Exposed pulp in carious dentine. Irreversibly diseased pulp. Allergy to the ingredients.

Side-effects: During application to exposed pulp, the strongly alkaline properties may cause pain reactions. Over filling of the root canal can cause troubles (sensitiveness to pressure and/or swelling), which regularly abate within 24-48 hours.

Interactions with other agents: In the treatment of deep carious lesions, calcium hydroxide neutralises the effect of acid-containing liners.

Dosage and mode of application:

Pulp capping (direct and indirect):

Small quantities of paste are placed on a mixing block and from there applied selectively to the exposed pulp or dentine adjacent to the pulp using an instrument. After application, the paste should be tamped using water-absorbing materials (pledgets of cotton or paper points). It is then sealed with a liner.

Temporary root filling:

The root canal has to be cleansed by hydrogen peroxide and Hypochlorite- SPEIKO® and to dry by sterile paper points. As a temporary filling material for root canals, SPEIKOCAL can be applied direct to the prepared canal using, for example; a lentula. If necessary, the application can be repeated. The calcium hydroxide paste should be removed before final root filling.

Treatment of deep carious lesions:

Before applying a thin coating of SPEIKOCAL, clean and drain cavity. If a thicker coating is desired, apply in layers. Briefly dry each layer in a gentle stream of air. Do not apply to the cavity margins, and remove all residue of paste. Then insert liner and filling.

Additional information: After use, make sure to recap the needle with the protective cap filled with calcium hydroxide (reservoir) so that no air will reach the product. The carbon dioxide in the air will otherwise cause the product to harden by way of conversion to calcium carbonate. If the paste has hardened in the protection cap, the protective function can be restored, if 1-2 drops of boiled water are added.

Storage: After usage, close the syringe by the protection cap immediately.

Special warnings:

Calcium hydroxide causes local irritation and corrosive effects on the skin, eyes and mucous membranes. Particles of calcium hydroxide in contact with the skin or eyes must be mechanically removed by appropriate means, then rinsed immediately and thoroughly with water. Do not use this medical product after the expiry date.

Keep this medicine out of reach of children.

Commercial presentation:

Syringe containing 2 ml paste and three disposable cannulas Article No. 1055

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SPEIKO®- Dr. Speier GmbH, P.O.Box 7204, D-48039 Münster