SPEIKO® Silikatzement (vormals Zhanelka)

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Pulver: Siliciumdioxid, Aluminiumoxid, Calciumoxid, Natriumoxid.

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Stand der Information: November 2005

SPEIKO® silicate cement (previously Zhanelka)

Composition:

Powder: silicon dioxide, aluminium oxide, calcium oxide, sodium oxide,

SPEIKO® Litho universal liquid for quick-setting: aluminium and zinc in phosphoric acid.

Applications:

SPEIKO® silicate cement is suitable for anterior tooth fillings in the visible area with Black class III and V cavities (lateral fillings without cutting edge involvement and cervical fillings.)

Contraindications:

Not suitable for buccal fillings and edge build-up in the frontal area. Drying-out of the filling. e.g. due to drying during dental treatment or to oral breathing by the patient, leads to structural defects (hairline cracking) and consequently to whitening of the filling.

Side effects / interactions

Usage for the intended purpose is not known to have induced any undesirable effects. The same applies to interactions with other agents.

Dosage instructions, form and duration of application

Mix powder in a volumetric ratio of 2.2:1 with SPEIKO® Litho universal liquid, adding the powder to the liquid and not vice versa. The mixing operation should be continued for ca 1 minute until a smooth, pasty consistency is obtained. Like other dental cements. SPEIKO® silicate cement is mixed on a glass plate, this should be cooled before preparing larger quantities to prolong the handling time. To prevent discolouring, a wear-resistant metal or agate spatula should be used. Pulp protection is essential for vital teeth (pulp necrosis may still occur after a long period). The cavity should be filled with one portion if possible, then covered immediately with matrix band and fixed under pressure pending primary hardening. The free filling surface must then be protected with clear varnish from drying out and from moisture penetration. To prevent irreparable damage to the varnishing, the patient should be instructed to wait 2 hours before eating. Polishing after not less than 24 hours. The setting time calculated from the start of the mixing procedure is 4 - 6 min. SPEIKO® Litho universal liquid is used and on hot examining temperature. Correct setting can be controlled if required by examining the residues on the glass plate.

Special instructions:

The liquid contains phosphoric acid and has a corrosive effect. In the event of inadvertent contact with the skin, mucosa or eyes, rinse immediately with plenty of water. Lack of appropriate pulp protection may result in pulp necrosis. The patient should not eat for the first 2 hours after receiving the filling. This medical product should not be used after the marked date. Keep out of reach of children.

Available in the following packs:

SPEIKO® Silicate cement Powder, shade 2, light yellow

Contents: 40 g Art.-no. 1430

SPEIKO® Litho universal liquid quick-setting

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