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**Danger**  
Liquid // Flammable liquid and vapor. Toxic in contact with skin or if inhaled. Harmful if swallowed. Causes serious eye irritation. Causes skin irritation. May cause an allergic skin reaction. May damage the unborn child. Suspected of damaging fertility. Suspected of causing cancer. May cause respiratory irritation. May cause damage to organs through prolonged or repeated exposure. Powder // May form combustible dust concentrations in air. Toxic if inhaled. May cause an allergic skin reaction. May cause cancer. Suspected of damaging fertility or the unborn child. Suspected of causing genetic defects. May cause damage to organs through prolonged or repeated exposure. Refer to product SDS for additional health, safety and disposal information.

⚠ This product can expose you to chemicals including Cadmium, Titanium Dioxide, N,N-dimethyl-p-toluidine, and dibutyl phthalate, which are known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

Flashpoint 45°C (113°F)

KeyStone Industries  
52 West King St  
Myerstown, PA 17067 USA  
800.333.3131  
keystoneindustries.com

CE  
KeyStone Europe LLC  
Botanweg 7  
53498B Oss Netherlands

CAUTION: US Federal law restricts this device to sale by, or on the order of a dental professional.

MD Medical Device  
keystoneindustries.com/symbolglossary

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### Bosworth Trim/Trim II Instructions for Use

Trim and Trim II are fast-setting, non-irritating PEMA temporary crown and bridge materials.

#### DIRECTIONS

1. Take an impression with material of choice. (Recommended: Bosworth Supergel)
2. Complete tooth preparations. Take another impression from which the final crown or bridge is to be made. Match tooth shade to closest Trim or Trim II shade.
3. Cut peripheral border and interproximal areas for best fit.
4. Mix powder and liquid until honey-like consistency. Approximate powder to liquid ratio is 6.3g of powder to 3ml of liquid. This amount would make 9 full jacket crowns.
5. Fill each tooth area within the impression where preparatory work has been done with Trim acrylic.
6. When Trim gets dull, reset the impression to return to its original position.
7. Leave in place for 30-90 seconds. Remove when first feels warm. When material is rubbery it may be removed from impression and trimmed. May be placed back into impression until fully set.
8. Trim and polish as desired. Cement bridge or crown in place with a non-eugenol cement.

#### NOTES

1. Using Bosworth Glaze saves polishing time.
2. This product generates heat during polymerization that could be harmful to the patient or dental professional. To avoid excessive heat or locking into undercut, take Trim out of mouth as soon as it cures to a rubbery rebound stage or when the patient senses the rising heat. Do not let Trim cure to completion in mouth.
3. May be bench cured or placed in pressure pot (Recommend: Bosworth Pressure Pot).

CAUTION: Certain individuals may be sensitive to products containing acrylic.

WORKING TIME: 3-4 minutes / 2-3 min (fast)  
TOTAL SET TIME: 8 minutes / 6 minutes (fast)