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Printed in Germany - Imprime en Allemagne

SPEIKO® - Dr. Speier GmbH
product information for the user
please read carefully

Palate patterns SPEIKO

Composition:

Our palate patterns consist of a flexible, transparent plastic which becomes malleable on heating.

Application:

They are used for upper dentures to show a replica of the natural palatine ridges and are re-usable.

Mode of application:

Make the pattern malleable by using hot air or hot water and place them onto the plaster model before setting up. For doing this the outer edges of the pattern can be cut to the size of the model. Heat again and press the pattern onto the model. Remove the pattern, put it aside and keep it in a safe place whilst setting up. Prepare the denture as usual using pink modeling wax. After the denture's wax trial secure the wax set-up onto the model with wax in a circular pattern. Lay the palate side of the prepared pattern onto the wax base, mark the outlines of the pattern and cut the wax base following the markings. Then insert the pattern into the set-up and neatly adhere wax. Invest the denture as usually and remove the pattern after boiling out.

Commercial product form:

Palata Patterns SPEIKO contents: 6 pcs. Art.-Nr. 1801

Information status: december 2009

SPEIKO-Dr. Speier GmbH, Walther-Rathenau-Straße 59, 33602 Bielefeld
Telefon 0521 770 107-0 , Telefax 0521 770 107-22 , e-mail: Info@speiko.de ,
www.speiko.de

Printed in Germany Imprime en Allemagne

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SPEIKO -Dr. Speier GmbH, Walther-Rathenau-Straße 59, 33602 Bielefeld