



CE 0482

Aptal-Resin Root Filling

Composition:

Powder contains zinc oxide, silicon dioxide (Aerosil), colophony, silver powder.
Liquid contains eugenol, colophony, turpentine oil, Peru balsam, olive oil.

Fields of application:

– Permanent closure of root canal with root canal posts

Contra-indications:

– In case of hypersensitivity to ingredients
– Direct contact with composite fillings has to be avoided; otherwise, curing cannot be warranted

Side effects/interactions:

None known if used according to intended purpose.

Dosage, type and duration of treatment:

Preparation:

Prepare, clean and dry the root canal to be filled prior to the application of the material.

Dosage and mixing:

Mix powder with liquid until the desired consistency is achieved.

Direct application into the root canal:

The paste can be placed in the root canal with the aid of the applicator. Subsequently, push in the root canal post to achieve a dense root filling.

Indirect application into the root canal:

The root canal post is wetted with Aptal-Resin Root Filling on the glass plate and subsequently inserted in the canal to achieve a dense root filling.

Removal:

As soon as Aptal-Resin Root Filling is cured, it can no longer be removed with conventional solvents. If Aptal Resin Root Filling is used with Guttapercha posts, the root canal filling can be removed in accordance with the specified techniques for the removal of Guttapercha.

Notes:

Aptal-Resin Root Filling may not be used after the expiration date.

Storage at room temperature.

Trading formats:

Aptal-Resin Root Filling	Content: 40 g	Art. no. 1003
Aptal-Resin Root Filling	Content: 10 ml	Art. no. 1004
Aptal-Resin Root Filling Set	Content: 10 ml/40 g	Art. no. 1002

Status of information: January 2017