

Speiko®- Dr. Speier GmbH
Product information for the user
Please read carefully



SPEIKOSEPT CHX 2 %-Gel

Composition: Gel contains 2 % chlorohexidin-diguconate

Applications:

Cleansing of contaminated root canal instruments during treatment of root canals

Dosage instructions, form and duration of application:

Pour a sufficient amount of gel into an appropriate vessel (endo tray boxes). Immerse used instruments in the gel after use. A reduction of the number of bacteria can be anticipated on account of the bactericidal mode of action of chlorohexidin-diguconate. The addition of surfactants to the product ascertains that remnants adhering from treatment are rapidly and completely detached from the instruments. Subsequent rinsing of the instruments is not necessary. Additional preparation of the instruments before sterilising is recommended.

Special instructions:

This medical product must not be used after its shelf-life has expired.
Close syringe tightly after use and store at room-temperature.
Keep this medical product out of the reach of children.

Available in the following packs:

Speikosept CHX 2%-Gel contents: 3 g Article-No. 1071

Status: december 2010

SPEIKO®- Dr. Speier GmbH P.O.Box 7204 D-48039 Münster
Telephone ++49/251/789040 Telefax ++49/251/7890422 E-Mail info@speiko.de Internet
www.speiko.de

Printed in Germany Imprimé en Allemagne