



## SPEIKOSEPT CHX 2 %

**Composition:** Solution contains 2% chlorhexidin-digluconat

**Applications:**

Cleansing of contaminated root canal instruments during treatment of root canals

**Dosage instructions, form and duration of application:**

Put a sufficient amount of solution into an appropriate vessel (endo tray boxes). Immerse used instruments in the solution after use. A reduction of the number of bacteria can be anticipated on account of the bactericidal mode of action of chlorhexidin-digluconate. Speikosept CHX 2% Solution is excellently suitable to eliminate the bacterium *Enterococcus faecalis*. (bacterial reduction 99%)

The addition of surfactans to the product ascertains that remnants adhering from treatment are rapidly and completely detached from the instruments. Subsequent rinsing of the instruments is not necessary. Additional preparation of the instruments before sterilising is recommended.

**Special instructions:**

This medical product must not be used after its shelf-life has expired.  
Close bottle tightly after use and store at room-temperature.  
Keep this medical product out of the reach of children.

**Available in the following packs:**

Liquid:	contents: 30 ml	Article-No. 1056
	contents: 100 ml	Article-No. 1070

Status: april 2011

---

SPEIKO® - Dr. Speier GmbH Postfach 7204 D-48039 Münster  
Telephone +49 (0)251 78904-0 Telefax +49 (0)251 78904-22 E-Mail: [info@speiko.de](mailto:info@speiko.de) [www.speiko.de](http://www.speiko.de)

Printed in Germany - Imprimé en Allemagne