



SPEIKO Prophylaxis powder soft with mint or lemon flavour

Composition:

Contains sodium carbonate, mint flavour or lemon flavour.

Fields of application:

- To remove deposits and plaque on tooth enamel
- For the preparation of the surface prior to cementing or gluing fillings, overlays, caps and veneer systems
- For the preparation of the surface prior to the application of composite restorations
- For the removal of plaque prior to the application of sealing agents and fluoride treatments
- For the cleaning prior to gluing orthodontic brackets and the effective removal of plaque and discolorations for patients under orthodontic treatment
- For the cleaning of implantation brackets prior to applying pressure
- For dental use only

Contra-indications:

- SPEIKO Prophylaxis powder soft is to be used exclusively for supra-gingival treatments
- Do not direct powder jet onto fillings, caps, bridges, dentine, root cement and demineralised enamel
- In case of hypersensitivity to ingredients

Side effects/interactions:

None known if used according to intended purpose.

Dosage, type and duration of treatment:

The principal of working from red to white has to be observed.

The gentle, less abrasive cleaning is possible due to the reduced corn size compared to SPEIKO Prophylaxis powder normal.

SPEIKO Prophylaxis powder soft is suitable for all conventional powder jet devices. Use SPEIKO Prophylaxis powder soft in accordance with the instructions of the device manufacturer!

The working time with the powder jet should be kept to a minimum.

The regulations regarding the suction technique by the device manufacturer have to be observed.

Notes:

SPEIKO Prophylaxis powder soft may not be used after the expiration date.

Storage at room temperature.

Trading formats:

SPEIKO Prophylaxis powder soft with mint flavour	Content: 200 g	Art. no. 1023
SPEIKO Prophylaxis powder soft with lemon flavour	Content: 200 g	Art. no. 1024

Status of information: February 2017