



CE 0482

## SPEIKOCOPAL

### Composition:

Contains 12.5% copal resin, diethyl ether, ethanol.

### Fields of application:

- For the lining of shallow cavities
- As protective paint for silicate and silico phosphate cement fillings
- Only for dental use

### Contra-indications:

- In case of utilisation of plastic fillings
- In case of usage of glass ionomer and polycarboxylate cements
- In case of hypersensitivity to ingredients

### Side effects/interactions:

None known if used according to intended purpose.

### Dosage, type and duration of treatment:

Following cleaning and drying of cavity, the lacquer is applied in a thin layer and dried with gentle air flow. If a second layer of the cavity lacquer is to be applied, the second layer can only be applied once the first is completely dry.

The cavity rim has to be free from cavity lacquer prior to the application of the filling material.

### Notes:

Diethyl ether is a very volatile, highly inflammable liquid. Storing in the brown, light-proof bottle.

SPEIKOCOPAL may not be used after the expiration date.

Storage at room temperature.

### Trading formats:

SPEIKOCOPAL                      Content: 20 ml                      Art. no. 1049

Status of information: January 2017