

SPEIKO® - Dr. Speier GmbH

Package leaflet

Please read carefully



**SPEIKOPLAQUE /  
SPEIKOPLAQUE 3% / SPEIKOPLAQUE DUO**  
Staining liquid for the visualisation of dental plaques



**Composition:**

SPEIKOPLAQUE and SPEIKOPLAQUE 3% contain erythrosin (E 127), purified water.  
SPEIKOPLAQUE DUO contains Patent Blue dye (E 131), Phloxin B; purified water.

**Applications:**

Indicator liquid to visualise plaque on dental surfaces. Plaques assume a more or less red colour after application of SPEIKOPLAQUE or SPEIKOPLAQUE 3%, whereas SPEIKOPLAQUE 3% colours more intense.  
After application of SPEIKOPLAQUE DUO assume older plaque blue colour, younger plaque red colour.

**Contraindications:**

Allergic reactions to erythrosin, iodine and sunlight (photodermatosis).  
Hyperkinetic syndrome in children (SPEIKOPLAQUE and SPEIKOPLAQUE 3%).  
Children under six years of age (SPEIKOPLAQUE and SPEIKOPLAQUE 3%).

**Side-effects/interactions:**

Unknown.

**Dosage instructions, mode and duration of application:**

Add several drops of the indicator liquid into a Dappen-dish. The liquid is applied to the dental surface with a brush or cotton swab. Already existent plaques become immediately visible by red (SPEIKOPLAQUE DUO also blue) colour. Removal of the stain is accomplished by rinsing with water or teethbrushing.

**Special instructions:**

Do not use this medical product after its shelf-life has expired.  
Keep out of the reach of children.

**Available in the following packs:**

liquid	SPEIKOPLAQUE	Contents: 20 ml	Art. No.: 1052
liquid	SPEIKOPLAQUE 3%	Contents: 20 ml	Art. No.: 1075
liquid	SPEIKOPLAQUE DUO	Contents: 20 ml	Art. No.: 1054

Status: october 2015

Printed in Germany · Imprimé en Allemagne