



CE 0482

SPEIKO® Universal Liquid Litho
(previously Zhanelka)
normal setting/fast setting

Composition:

Contains phosphoric acid, aluminium ions and zinc ions.

Fields of application:

Liquid to mix the following dental cement powder:

- SPEIKO® Silicate Cement (previously Zhanelka Siliceous Cement)
- SPEIKO® Zinc Porcelain Cement opaque (previously Zhanelka Zink Porcelain Cement opaque)
- SPEIKO® Special Copper Cement opaque (previously Zhanelka Special Copper Cement opaque)
- SPEIKOLIT® (previously Zhanelkalit)

The contra-indications, side effects/interactions and the dosage, type and duration of the application are specified in the operating instruction of the powder.

Notes:

SPEIKO® Universal Liquid Litho contains phosphoric acid, which may cause chemical burns.

In case of accidental contact with skin, mucous membranes or eyes, rinse with water immediately.

SPEIKO® Universal Liquid may not be used after the expiration date.

Storage at room temperature.

Trading formats:

SPEIKO® Universal Liquid Litho normal setting

Content: 20 ml

Art. no. 1437

SPEIKO® Universal Liquid Litho fast setting

Content: 20 ml

Art. no. 1441

Status of information: January 2017
