



CE 0482

# Zinc Oxide-Eugenol Special Paste

normal setting / fast setting

## Composition:

Powder contains zinc oxide, zinc acetate, colophony, talcum, silicon dioxide.  
Liquid contains clove oil, in the ZOE Special Paste fast curing additionally olive oil.

## Fields of application:

- Interim treatment of cavities, particularly in case of caries profunda-treatments
- Interim fillings for endodontic treatments
- As thermo-insulating under fill for cement and metal fillings

## Contra-indications:

- In case of hypersensitivity to wood resin and clove oil (Eugenol)
- With the use of composite and compomer fillings
- At opened pulp or near-pulp areas

## Side effects/interactions:

None known if used according to intended purpose.

## Dosage, type and duration of treatment:

The paste is mixed, whereby 1 drop of liquid binds approx. four times the amount of powder. Within a few seconds, the paste becomes softer directly after mixing and a little more powder can be added. It should be mixed to a firm texture. The paste can be easily applied into the dry cavity and pressed in with a moistened cotton swab. If necessary, additional paste can be added. Curing commences shortly thereafter under the effect of saliva and water. The curing is complete after approx. 24 hours.

## Notes:

The desired hardness can be influenced by varying the mixing ratio. Greater ultimate hardness is achieved with more liquid, lesser ultimate hardness with more powder. The patient should avoid excessive mastication until the paste has cured to the desired ultimate hardness. It is able to carry a full load after 24 hours.

Zinc Oxide-Eugenol Special Paste may not be used after the expiration date.

Storage at room temperature.

## Trading formats:

ZOE Special Paste Normal Setting	Content: 40 g	Art. no. 1082
ZOE Special Paste Normal Setting	Content: 10 ml	Art. no. 1087
ZOE Special Paste Normal Setting Set	Content: 40 g/10 ml	Art. no. 1086
ZOE Special Paste Fast Setting	Content: 40 g	Art. no. 1093
ZOE Special Paste Fast Setting	Content: 10 ml	Art. no. 1098
ZOE Special Paste Fast Setting Set	Content: 40 g/10 ml	Art. no. 1097

Status of information: January 2017