



CE 0482

CALASEPT-SPEIKO

with x-ray contrast

Composition:

Contains calcium hydroxide and barium sulphate in watery suspension, calcium chloride, potassium chloride, sodium chloride, sodium hydrogen carbonate, purified water.

Fields of application:

- Supply of caries-free, near-pulp dentin
- Temporary insertion in root canal
- Caries profunda treatment and/or indirect pulp capping
- Only for dental use

Contra-indications:

- In case of pulp exposure in carious dentin
- In case of inflammatory changes of pulp
- In case of hypersensitivity to ingredients

Side effects/interactions:

None known if used according to intended purpose.

Dosage, type and duration of treatment:

Pulp capping (indirect):

Small amounts of Calasept are placed on a mixing pad and from there applied with an instrument to the dentin near the pulp. Following the application, the past should be compressed with water-absorbing material (cotton pellets or paper tips). Subsequently, it is covered tightly with a lining material.

Temporary insertion in root canal:

Clean root canal with Hypochlorite-SPEIKO 1 % or Hypochlorite-SPEIKO 3 % or Hypochlorite-SPEIKO 5.25 % and dry with sterile paper tips. As a temporary filling for root canals, CALASEPT-SPEIKO can be inserted directly into the prepared canal with the aid of e.g. applicator. The application of the paste into the root canal is also possible with the aid of the enclosed disposable cannula. If necessary, the treatment can be repeated. CALASEPT-SPEIKO should be removed prior to the definite root filling.

Caries profunda treatment:

Clean and dry the cavity prior to applying CALASEPT-SPEIKO in a thin layer. If a thicker layer is required, please apply in layers. Briefly dry each layer with a gently air current. Subsequently, place lining and filling.

Notes:

Usage of correct cannulas: In order to prevent curing of the material, the disposable cannula has to be placed into the enclosed rubber stop and the bent cannula closed with the draw wire.

Usage of correct cartridge syringe: We recommend the usage of a non-self-aspiring cartridge syringe. The syringe used for CALASEPT-SPEIKO has to have a well-centred piston which is not laterally movable. Syringes with aspiration hooks or V-shaped pistons cannot be used.

CALASEPT-SPEIKO may not be used after the expiration date.

Storage at room temperature.

Summary reports on safety and clinical performance for the medical device are available in the European database for medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Trading format:

CALASEPT-SPEIKO, 2 x 2,5 g, 3 disposable cannulas	Art. no. 1005
25 disposable cannulas	Art. no. 1006

Status of information: September 2026